

Adjustable Bed Cradle

Code: 125

CHEVRON

This lightweight adjustable blanket cradle is designed to keep bedclothes clear of feet and legs.

Fits under mattress.

Adjustable height: 530mm-680mm.
Length: 500mm.
Width: 480mm.

The purpose of the cradle is to support the weight of the bedding from the patient's legs.

PARTS LIST:

Base Tubes (A)	x 2
Connecting Tubes (B)	x 2
Horseshoe Clips	x 4

TO ASSEMBLE:

1. Fit the two straight tubes (B) to the ends of the Base Tubes (A).
 2. Line up the holes in parts A and B and fit the horseshoe clips.
- Cradle is now assembled.

TO ADJUST:

To adjust the height of the cradle, remove horseshoe clips, slide part 'A' into part 'B' to give the desired height. Replace horseshoe clips.

ENSURE ALL HORSESHOE CLIPS ARE FULLY LOCATED

TO FIT TO BED:

One side of the cradle slides under the mattress and the other side supports the bed clothes over the user. It is usual for the cradle to be fitted at the foot of the bed but it can, if required, be fitted to the side of the bed.

CLEANING:

It is recommended that the cradle is regularly cleaned with a mild disinfectant.

DO NOT USE BLEACH OR ANY SOLVENTS (e.g. LIGHTER FLUID).

